Fito® cream

Fito® cream is a medical device consisting of water dispersible dermatological cream contained in an aluminium tube.

INGREDIENTS: Rigenase® (wheat extract), 2-phenoxyethanol, polyethylene glycol400, polyethylene glycol1500, polyethylene glycol3000, polyethylene glycol4000, liquid paraffin, cetyl alcohol, stearyl alcohol, glycerine, purified water.

PROPERTIES: Fito® cream forms a protective barrier against the external environment, thus creating favorable conditions for a fast and correct skin re-epithelialization and contributes to keeping its microenvironment under control; consequently, the product is indicated for the treatment of ulcers, sores, wounds, abrasions and first and second degree burns. Rigenase® is a specific wheat extract.

INSTRUCTIONS FOR USE: How to apply the product: When first using the product, pierce the tube by using the small punch contained in the upper part of the cap. Spread the necessary amount of cream on the area and cover with sterile gauze, possibly imbibed with more cream. If there are no contraindications, in following applications it is possible to previously cleanse the area with sterile water as the cream is totally water dispersible and leaves no residues. Depending on the seriousness, extension and location of the damage, applications can be reduced to simple massages to let the cream be absorbed. After use, close the tube by tightly screwing the cap on it. How often to apply the product: follow medical advice. Cutaneous use.

SIDE EFFECTS: No side effects have been reported. However, it is important to report the possible onset of side effects to the doctor or the pharmacist.

CONTRAINDICATIONS: Known or suspected hypersensitivity to any of the ingredients.

PRECAUTIONS FOR USE: The use of all topical products, especially if continued, can cause the onset of sensitization phenomena. Should this occur, the use must be stopped and, if necessary, appropriate treatment must be administered.

INTERACTIONS: There are no known interactions or incompatibilities with other products.

WARNING: Carefully read the enclosed leaflet before use. Keep out of children's reach and sight. Do not swallow and avoid contact with eyes. Do not use after the expiry date shown on the package; the expiry date refers to the product properly stored in its unopened container. Do not use if the container has been damaged.

STORAGE CONDITIONS:
Do not store above 30°C. Do not freeze.

PACKAGING: 32 g tube and 10 g tube

DISPOSE OF THE CONTAINER RESPONSIBLY AFTER USE

DISTRIBUTED BY:
LEO Pharma A/S
Ballerup, Denmark

MANUFACTURED BY:
Farmaceutici Damor S.p.A.
Via E. Scaglione, 27 - Naples - Italy
Fax: +39 0817405172
e-mail:damor@farmadamor.it

0425

Date of information leaflet revision: March 2020

*Trademark

Description of the symbols used on primary and secondary packaging:

Dispose of the container responsibly after use.

Stamp of compliance with EC directives.

Limit of temperature of storage conditions.

Carefully read the information leaflet.

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